



Kansas Health & Environmental Laboratories  
SE Dwight St., Topeka, KS 66620

Phone (785) 296-1620 Fax (785) 296-0978  
CLIA #17DO648254

### BLOOD LEAD COLLECTION FORM INSTRUCTIONS

1. **PRINT** legibly using dark ink. **DO NOT** use pencil. Photocopied forms are not acceptable.  
.This form may be used for collecting filter paper blood spots **OR** submitting whole blood specimens.  
.Complete all requested information and **PRINT** in capital letters. Ensure letters and numbers are inside boxes: **A B C 1 2 3**

Sex: M  F

- .Where circles are provided, fill in completely to indicate selection :  
.Be sure to fill in the Facility ID with the unique number assigned to your facility by KHEL. Also put the name and address of your facility.  
.Locate **Collection Type** box and indicate the type of collection Venous or Filter paper. Please fill in only one circle.

#### Filter Paper Blood Collection:

- .Collect specimen as indicated on filter paper coverlet.
- .Place collection form in an envelope and mail completed form to KDHE/KHEL Labs using address label provided on the front of the form.
- .When placing more than one specimen in an envelope, alternate forms so that blood spots on adjacent forms are not in contact.

#### Whole Blood Specimen Collection:

- .Collect whole blood specimen using an approved EDTA Vacutainer tube.
- .Attach the peel-off 'tube label' from the front of the form to whole blood vacutainer tube.
- .Mail the form and labeled specimen tube to KDHE/KHEL Labs using the address label provided on front of the form.

**DO NOT ALLOW BLOOD SPOTS TO COME INTO CONTACT WITH COVERLET OR ANY OTHER SURFACE UNTIL DRY**

**Filter paper compliant with CLSI Standard LA 4**



IVD

Fold back wrap around cover before collecting filter paper blood lead specimen.  
The blood spots must be covered to meet U.S. Postal requirements  
DO NOT cover blood spots until completely dry.  
(1) Do not touch sample area  
(2) Do not use if damaged